

## SciCan BRAVO G4 Quick Reference Guide


Operator's Manual (located on the USB key) should be consulted before using this autoclave.


### +49(0)756198343-0


#### **BASIC OPERATION**

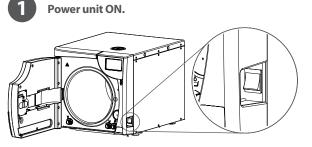


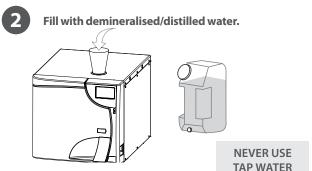

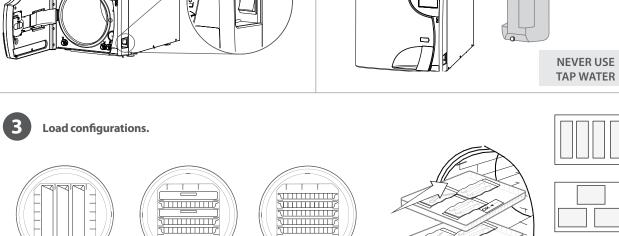

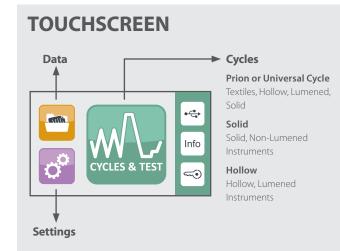


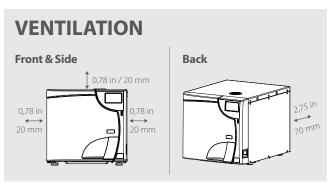


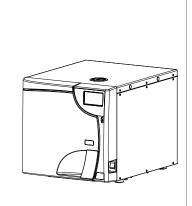



Close unit.




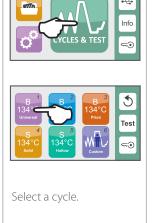


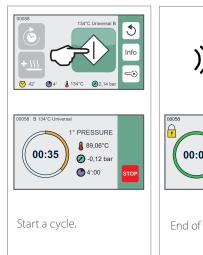


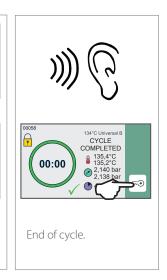


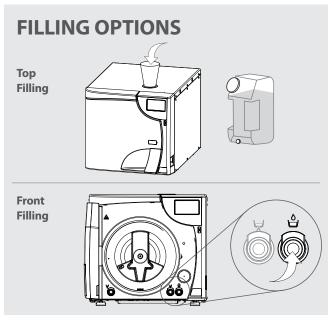

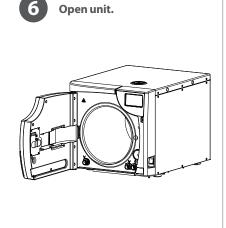

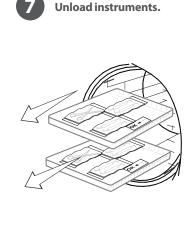

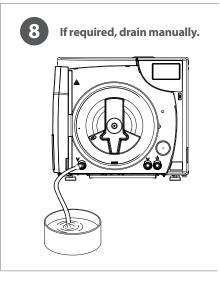

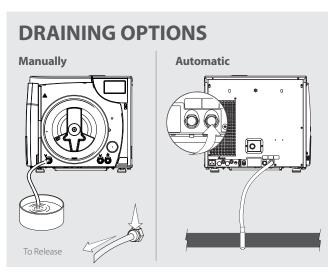

#### **MAINTENANCE SCHEDULE**

.,	
Daily	Clean the <b>door gasket</b> and internal part of the door with a clean wet cloth. Check for wear and replace if required.
	Completely empty the <b>waste tank</b> before refilling the water reservoir.
Weekly	Clean the <b>chamber and appropriate accessories</b> with a wet cloth. If required, use a neutral detergent (without alkalines and bleaches). Carefully rinse with demineralized/distilled water and remove any residue.
	Inspect and clean chamber <b>anti-dust filter</b> . Replace if necessary.
	Ensure area around BRAVO G4 is dust free.
Monthly	Drain & clean the <b>internal water reservoir</b> as per the instructions outlined in the Operator's Manual.
	If applicable, clean the <b>external water tank</b> used for

	Check the <b>bacteriological filter</b> and replace as required.  Lubricate the <b>door lock</b> . Using a clean cloth, remove any residues and lubricate the inside of the bushing with a film of the silicone-based grease provided.
Every 3-6 Months	Inspect and clean the chamber drain filter.  Replace if required.
Year or 1,000 Cycles	Replace the <b>door gasket</b> .

Spare Parts and Accessories		
97400145	Door Gasket 17L/22L	
97467176	Door Gasket 28L	
97290160	Bacteriological Filter	
97290210	Chamber Drain Filter	
99950390	Chamber Rack 17L	
99950317	Chamber Rack 22L	
99951041	Chamber Rack 28L	
99950394	Pouch Rack	
99951317	Tray 17L	
99951318	Tray 22L	
99951042	Tray 28L	
97467066	Tray Extractor	

# IMPORTANT! Users of this device must be sure to select the most appropriate sterilisation cycle according to the recommendations of the leading infection control authorities and local regulatory

guidelines / recommendations.

automatic filling.