SciCan STATCLAVE G4 Quick Reference Guide

Operator's Manual should be consulted before using this autoclave.

BASIC OPERATION

1

Power unit ON.

2

Fill BOTH reservoirs with distilled water.

3

Load configurations.

For unwrapped, space evenly.

For pouches, use pouch racks provided.

For wrapped cassettes, use drying plates provided.

4

Close and latch door.

5

Select cycle.

6

Press START icon.

7

When cycle is complete, press STOP, open door and remove load.

TOUCHSCREEN

Solid / Unwrapped

Solid / Wrapped

Hollow / Unwrapped

Hollow / Wrapped

More Cycles

- Textiles / Porous
- Rubber & Plastic
- Custom Cycle

Test Cycles

- Vacuum Test
- Bowie-Dick Test

Network Connection

USB

Door Lock

Settings

UNLOCK DOOR

FILLING OPTIONS

Manual fill from top

Manual fill from front

Automatic fill

Make sure your Water Filling setting is set to AUTOMATIC.

DRAINING OPTIONS

Direct-to-Drain

Hardware provided with unit.

Waste Bottle

Check and empty regularly, every 4 cycles.

MAINTENANCE SCHEDULE

Daily	Wipe the door gasket clean with a damp, lint-free cloth. Clean external surfaces with a damp, lint-free cloth.
Monthly or 100 Cycles	<div>Drain and clean both water reservoirs and water reservoir filters. </div> <div>Inspect and clean the three chamber filters. </div> <div>Clean the external distilled water tank - if installed.</div> <div>Clean the chamber rack and trays.</div>
Weekly	Clean the chamber and, if applicable, the waste bottle. Disinfect external surfaces.
Every 6 Months or 500 Cycles	Perform all the cleaning tasks listed in the monthly schedule. Replace the bacteriological filter. Replace the door seal.
Every Year or 1,000 Cycles	A complete maintenance of the autoclave (by a SciCan-approved technician) is recommended.

Spare Parts / Consumables	
01-115481S	Door Seal, STATCLAVE G4
01-115479S	Bacteriological Air Filter
01-115478S	Chamber Filter Kit (3 filters)
01-115487S	Chamber Rack
01-115482S	Mesh Tray (1 pc)
01-115483S	Pouch Rack (1 pc)
01-115485S	Tray Extractor with Door Unlock Pin (in handle)
01-115484S	Drying Plate (1 pc)
01-115480S	Filter for reservoirs (1 pc)
01-115490S	Exhaust Tubes - 13 ft/4 m length (2 pcs)
USB-STICK2	USB Stick
01-115373	Manual Fill Container with Quick Connect Tubing for front fill option
01-115374	Automatic Fill Pump

IMPORTANT!

Users of this device must be sure to select the most appropriate sterilization cycle according to the recommendations of the leading infection control authorities and local regulatory guidelines / recommendations.

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