
SciCan STATCLAVE G4 Quick Reference Guide

Operator's Manual should be consulted before using this autoclave.



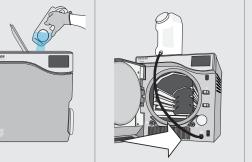



MAINTENANCE SCHEDULE

Wipe the door gasket clean with a damp, lint-free cloth.

setting is set to AUTOMATIC.

Clean the chamber and, if applicable, the

Spare Parts and Accessories01-115481SDoor Seal, STATCLA

5481S	Door Seal, STATCLAVE G4
5/709	Bactoriological Air Filtor

Daily

Clean external surfaces with a damp, lint-free cloth.

М	Monthly or	Drain and clean both water reservoirs and water reservoir filters.	Ev Mo 5 Cy
100	100 ycles	Inspect and clean the three chamber filters.	Ev Yo 1,i Cy
		Clean the external distilled water tank - if installed.	
		Clean the chamber rack and trays.	

_	Weekly	waste bottle.
_	,	Disinfect external surfaces.
-		
		Perform all the cleaning tasks listed in the monthly schedule.
	Every 6 Months or 500 Cycles	Replace the bacteriological filter.
		Replace the door seal.
	Every Year or 1,000 Cycles	A complete maintenance of the autoclave (by a SciCan-approved technician) is recommended.

01-115479S	Bacteriological Air Filter
01-115478S	Chamber Filter Kit (3 filters)
01-115487S	Chamber Rack
01-115482S	Mesh Tray (1 pc)
01-115483S	Pouch Rack (1 pc)
01-115485S	Tray Extractor with Door Unlock Pin (in handle)
01-115484S	Drying Plate (1 pc)
01-115480S	Filter for reservoirs (1 pc)
01-115490S	Exhaust Tubes - 13 ft/4 m length (2 pcs)
USB-STICK2	USB Stick
01-115373	Manual Fill Container with Quick Connect Tubing for front fill option
01-115374	Automatic Fill Pump

IMPORTANT!

Users of this device must be sure to select the most appropriate sterilization cycle according to the recommendations of the leading infection control authorities and local regulatory guidelines / recommendations.

www.scican.com



95-115214 NA EN FR R3. STATCLAVE is a trademark of SciCan Ltd. Manufactured by **SciCan Ltd.** 1440 Don Mills Road, Toronto, Ontario, Canada, M3B 3P9.