

STATCLAVE™ G4

Chamber Autoclave



Cutting Edge Sterilization Technology

STATCLAVE G4 is a chamber sterilizer that incorporates both pre-sterilization vacuum to improve steam penetration and post-sterilization vacuum to improve drying. Its 11" chamber and vacuum-assisted closed-door drying efficiently delivers sterilized and dried wrapped instruments.



Fast

STATCLAVE can efficiently sterilize and dry wrapped instrument loads in as little as 38 minutes.*



Dry

Closed-door drying and vacuum technology results in perfectly dried instruments every time.



Capacity

STATCLAVE's 11" chamber can sterilize up to 4 full-size cassettes and 4 exam cassettes, or up to 20 pouches with included pouch racks..




Gentle

Fresh water is used in every cycle, reducing the buildup of oils and debris.



G4 Technology

G4 equipment can connect and send information to your smart devices so you can have cycle data, instructions, and maintenance notifications at your fingertips.

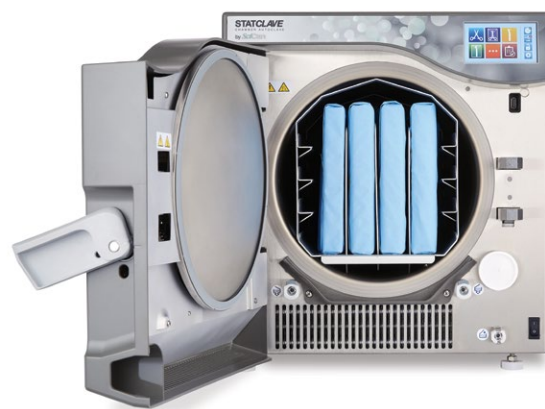

Versatile

STATCLAVE offers a number of filling & draining options, including easily accessible and cleanable top reservoirs, front manual fill and a rear automatic fill option that can be connected to the VistaPure water filtration system for automated filling.



Warranty

Reliable Protection, Guaranteed

Every STATCLAVE G4 is backed by a 2-year warranty. We stand behind the design and quality of our sterilizers, ensuring reliable performance for your practice.

*Results based on testing with a 3.5 lbs (1.59 kg) load.

Cycle Times and Maximum Load Weights

Cycle	Temperature / Time	Total Cycle Times (120V)*	Total Cycle Times (240V)*	Maximum Load Weight
Solid Unwrapped	270°F (132°C) / 4 mins	18 mins	17 mins	13.2 lbs (6 kg)
Solid Wrapped	270°F (132°C) / 4 mins	44 mins	41 mins	13.2 lbs (6 kg)
Hollow Unwrapped	270°F (132°C) / 4 mins	23 mins	19 mins	13.2 lbs (6 kg)
Hollow Wrapped (120V)	270°F (132°C) / 4 mins	50 mins	–	13.2 lbs (6 kg)
Hollow Wrapped (208V-240V)	270°F (132°C) / 4 mins	–	43 mins	7.7 lbs (3.5 kg)
Textiles/Porous	270°F (132°C) / 4 mins	49 mins	36 mins	4.4 lbs (2 kg)
Rubber & Plastic Unwrapped	250°F (121°C) / 20 mins	36 mins	31 mins	4.4 lbs (2 kg)

*Cycle time will vary depending on instrument load

Unit Specifications

Unit size (L x W x H) [†]	25" x 17.75" x 19.5" (635 x 450 x 495mm)
Volume	6.87 gal (26 L)
Chamber Dimensions (D x L)	11" diameter, 15" depth (297 x 381mm)
Weight	135.8 lbs (61.6 kg) (without water and packaging)
Voltage	120V, 60Hz, 12A / 208V-240V, 60Hz, 12A

[†]On the right side of the unit, allow 2" (50mm) of space for ventilation. Allow for at least 7" (180mm) above unit to enable access to reservoirs (if less than 7" screws on hinges can be removed to allow reservoir lid to slide in and out of position).

Recommended Accessories



STATPURE™ Water Filtration

Produces two types of high purity process water for sterilizers, automated instrument washers, dental bottles, and ultrasonic cleaners.

SWP400



DrainDock®

A multi-port drain adapter to create a unified drain connection point for steriCenter equipment.

- DD158** DrainDock 8-Port
- DD158FK** DrainDock 8-Port with fittings
- DDFK1** DrainDock Fittings Kit

www.scican.com

Manufactured for:
Dent4You AG
Bahnhofstrasse 2
CH-9435 Heerbrugg

Distributed by:
Coltène/Whaledent Inc.
235 Ascot Parkway
Cuyahoga Falls, OH 44223 US

Distributed by:
SciCan Ltd.
1440 Don Mills Rd.
Toronto, ON M3B 3P9, Canada

STATPURE and DRAINDOCK are manufactured by:
Vista Water Group
1244 CR 1475,
Ashland, OH 44805, USA

SciCan is a Coltène Group company. STATCLAVE is a trademark of SciCan Ltd., used under licence by Dent4You AG. DrainDock is a registered trademark of Vista Water Group. STATPURE is a trademark of Coltène/Whaledent AG.

SciCan | Your Infection
Control Specialist