

SciCan STATCLAVE G4

Chamber Autoclave



Cutting Edge Sterilization Technology

STATCLAVE G4 is a Class B vacuum sterilizer that incorporates both pre-sterilization vacuum to improve steam penetration and post-sterilization vacuum to improve drying. Its 11" chamber and vacuum-assisted closed-door drying efficiently delivers sterilized and dried wrapped instruments.



Fast

STATCLAVE can efficiently sterilize and dry wrapped instrument loads in as little as 38 minutes.*



Dry

Closed-door drying and vacuum technology results in perfectly dried instruments every time.



Capacity

STATCLAVE's 11" chamber can sterilize up to 4 full-size cassettes and 4 exam cassettes, or up to 20 pouches with included pouch racks.



G4 Technology

SciCan's G4 equipment can connect and send information to your smart devices so you can have cycle data, instructions, and maintenance notifications at your fingertips.



Gentle

Fresh water is used in every cycle, reducing the buildup of oils and debris.



Versatile

STATCLAVE offers a number of filling & draining options, including easily accessible and cleanable top reservoirs, front manual fill and a rear automatic fill option that can be connected to the VistaPure water filtration system for automated filling.

*Results based on testing with a 3.5 lbs (1.59 kg) load.

Cycle Times and Maximum Load Weights

Cycle	Temperature / Time	Total Cycle Times (120V)*	Total Cycle Times (240V)*	Maximum Load Weight
Solid Unwrapped	270°F (132°C) / 4 mins	18 mins	17 mins	13.2 lbs (6 kg)
Solid Wrapped	270°F (132°C) / 4 mins	44 mins	41 mins	13.2 lbs (6 kg)
Hollow Unwrapped	270°F (132°C) / 4 mins	23 mins	19 mins	13.2 lbs (6 kg)
Hollow Wrapped (120V)	270°F (132°C) / 4 mins	50 mins	–	13.2 lbs (6 kg)
Hollow Wrapped (208V-240V)	270°F (132°C) / 4 mins	–	43 mins	7.7 lbs (3.5 kg)
Textiles/Porous	270°F (132°C) / 4 mins	49 mins	36 mins	4.4 lbs (2 kg)
Rubber & Plastic Unwrapped	250°F (121°C) / 20 mins	36 mins	31 mins	4.4 lbs (2 kg)


*Cycle time will vary depending on instrument load

Unit Specifications

Unit size (L x W x H) [†]	25" x 17.75" x 19.5" (635 x 450 x 495mm)
Volume	6.87 gal (26 L)
Chamber Dimensions (D x L)	11" diameter, 15" depth (297 x 381mm)
Weight	135.8 lbs (61.6 kg) (without water and packaging)
Voltage	120V, 60Hz, 12A / 208V-240V, 60Hz, 12A

[†]On the right side of the unit, allow 2" (50mm) of space for ventilation. Allow for at least 7" (180mm) above unit to enable access to reservoirs (if less than 7" screws on hinges can be removed to allow reservoir lid to slide in and out of position).

Recommended Accessories

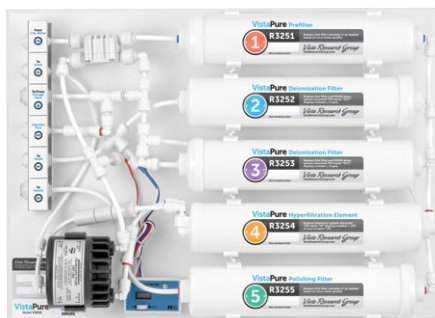

VistaPure

Water Filtration

Provides two grades of high-quality water for autoclaves, dental bottles, ultrasonic cleaners and the rinse cycle of instrument washers.

V3000



DrainHub

Multi-port Drain Adapter

The patent-pending device accepts up to eight drain connections in one small hub; saving space, cost, and time.

DH154 DrainHub 4 Hole

DH158 DrainHub 8 Hole



COLTENE International Dental Group

Manufactured for:

Dent4You AG

Bahnhofstrasse 2
CH-9435 Heerbrugg

Distributed by:

SciCan Ltd.

1440 Don Mills Rd.
Toronto, ON
M3B 3P9 Canada
T +1 416-445-1600
customerservice@scican.com

Distributed by:

Coltène/Whaledent Inc.

235 Scot Parkway
Cuyahoga Falls, OH
44223 US
T +1 330-916-8800
service@coltene.com

VistaPure and DrainHub are manufactured by:

Crosstex International, Inc.

1733 W. Parkside Lane
Phoenix, AZ 85027 / USA
T +1 800-531-2221
VistaResearchGroup.com

www.scican.com

STATCLAVE is a trademark of SciCan Ltd., used under licence by Dent4You AG. All Vista product names are trademarks of HuFriedy Mfg. Co. LLC, its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.

COLTENE