


# OPTIM BLUE Quick Reference Guide

The product's label should be consulted before using this surface disinfectant.


## Procedure for disinfecting elastomeric impression materials with OPTIM BLUE Liquid




**1** Once the impression is taken, rinse it well under running tap water to remove blood and saliva.

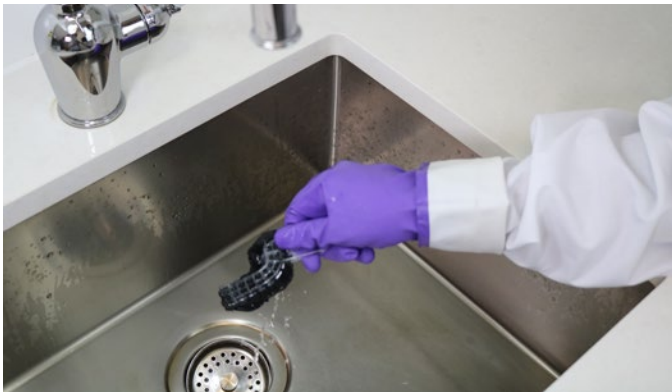

**2** After rinsing, gently shake the impression within the sink basin to remove any adherent water with minimal splatter.

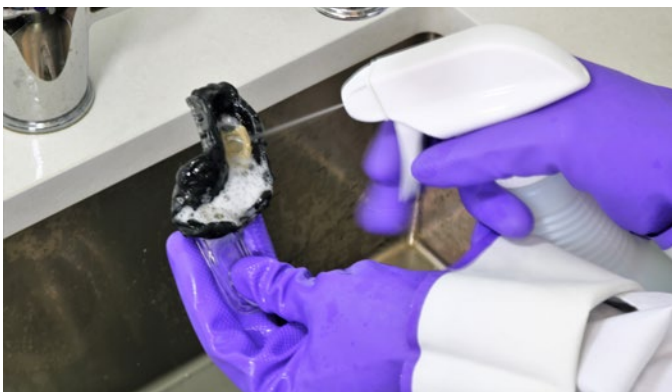

**3** Spray the entire impression with OPTIM BLUE. Use the contact times as stated on the label.

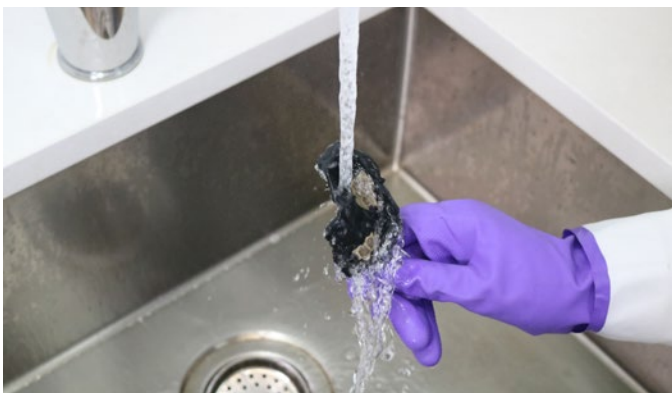

**4** Rinse the impression thoroughly with tap water to remove any excess disinfectant. After a thorough rinse, gently shake the impression within the sink basin to remove any adherent water with minimal splatter.



**5** Once dry, the impression is ready for further processing.



### OPTIM BLUE is safe to use

No caution, danger or warning required on labels. OPTIM is non-irritating to skin, eyes and respiratory system.

OPTIM BLUE has been tested in accordance with regulatory requirements as a One-Step Cleaner Disinfectant.

Please refer to the original container for complete directions and additional information.