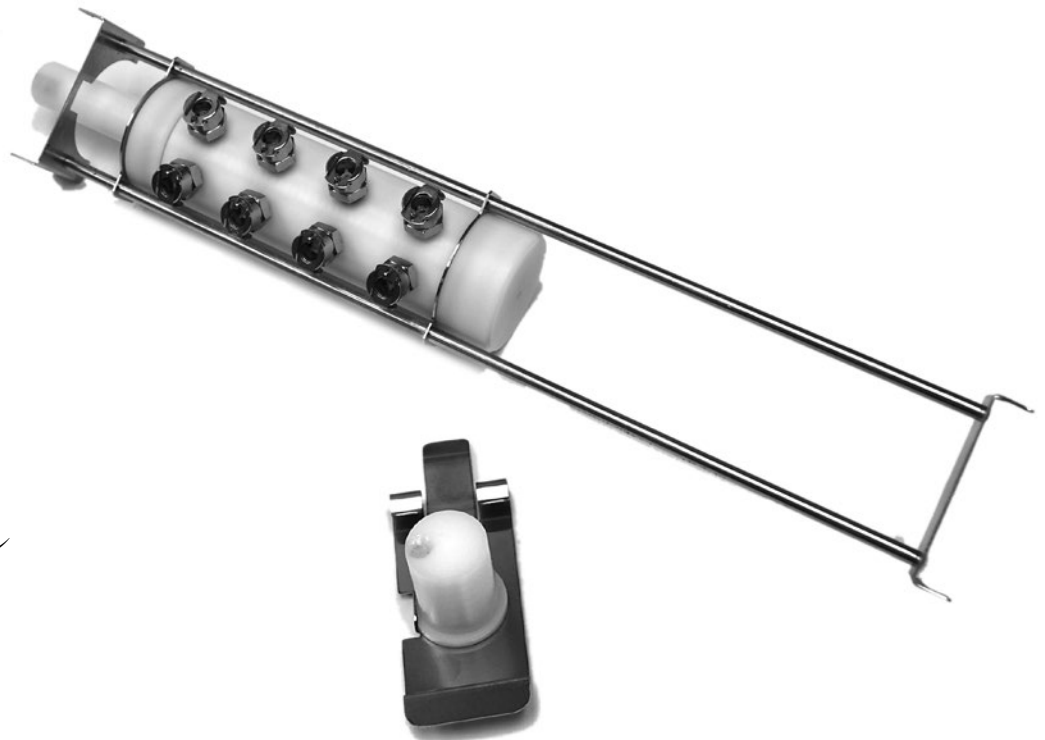
SciCan

HYDRIM[®] C61WD G4

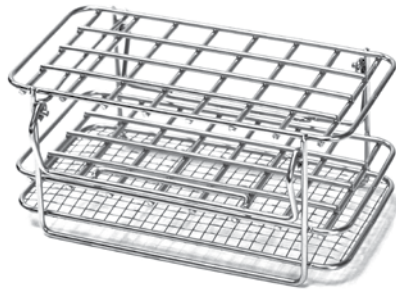
HANDPIECE CLEANING SYSTEM (LCS)

- Operator's Manual
- Mode d'emploi
- Bedienerhandbuch
- Manuale d'uso
- Manual del operador
- オペレーターマニュアル
- Brugermanual
- Bedieningshandleiding
- Instrukcja obsługi
- Manual de Utilização
- Manual de utilizare
- Инструкция по эксплуатации
- Användarmanual
- Керівництво з експлуатації

HYDRIM® C61WD G4 Lumens Cleaning System (LCS)


The Lumens Cleaning System (LCS) Adaptor can only be used with HYDRIM C61WD G4 models equipped with a lumens cleaning system. If your HYDRIM is LCS enabled, you will see a white plug on the inside back left of the chamber.

Rack for vertical instruments (optional)

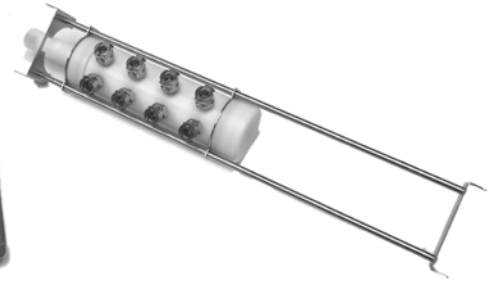
LCS Plug (included)


LCS Adaptor and Rack (included)

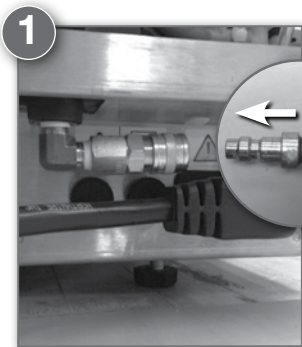Setting up the LCS unit

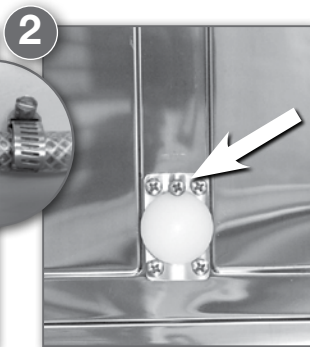
1 Pull the quick connect fitting ring back, insert air line coupling and release the ring to lock.

2 Remove the screw and pull out the white plug on the back wall of the unit's chamber.

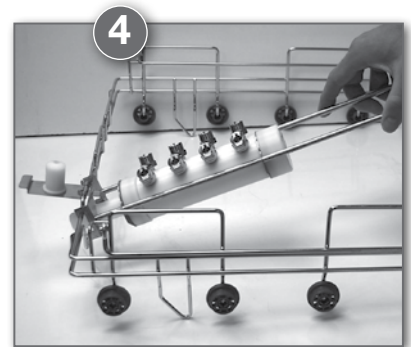
3 Attach the LCS plug to the HYDRIM trolley. (Stays in the down position when LCS rack is not being used)


4 Attach the LCS adaptor and rack to the trolley, engaging the tab at the top of the LCS plug. Place the cannula cleaning basket on the rails of the LCS rack.

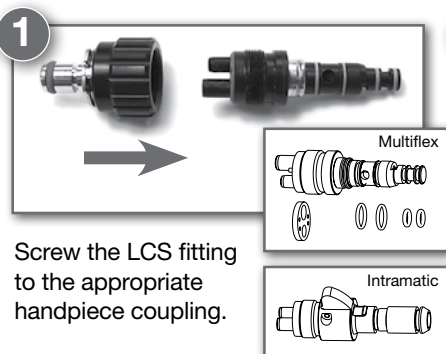
Connecting handpieces to the LCS adaptor

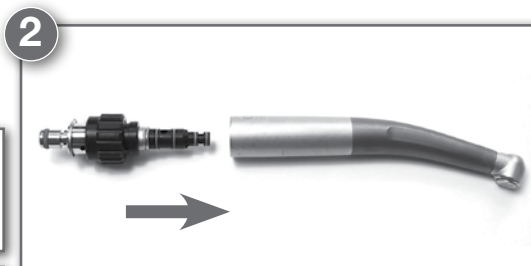1 Screw the LCS fitting to the appropriate handpiece coupling.

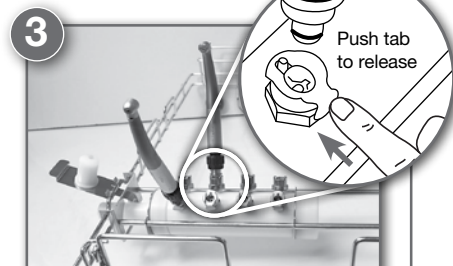2 Insert the handpiece coupling into the handpiece.




3 Push the handpiece into one of the ports.

Using the touchscreen menu

The HYDRIM will automatically detect the presence of the LCS adaptor. Select the desired program and follow the prompts. Upon completion, open the door and remove the handpieces from the adaptor by pushing on the tab at the side of the port.

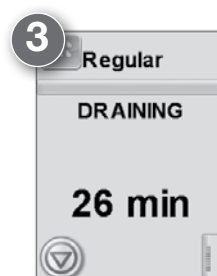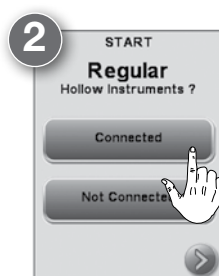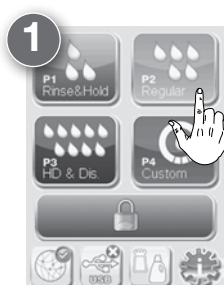Example: Regular cycle

Please note: As recommended by leading manufacturer/suppliers of handpieces, the hollow parts of instruments should be dried immediately after cleaning and disinfection using medical compressed air before additional processes or sterilization. This is to be followed by recommended regular maintenance procedures and application of maintenance agents/oils. Please follow all applicable health & safety regulations for your region.

Maintenance and troubleshooting

CLEANING THE LCS WATER FILTER

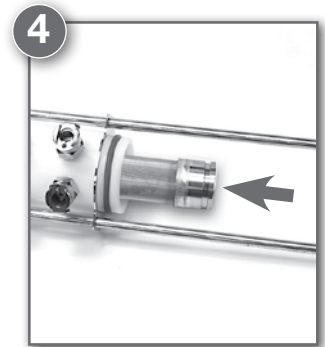
1
Unscrew the cap at the front of the LCS adaptor cylinder.


2
Remove the filter.

3
Rinse the filter and, if required, clean with a soft brush or using compressed air. Check and clean the O-ring. Remove any debris inside the housing.

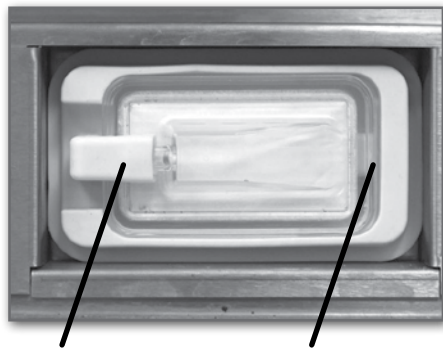4
Return the filter to the housing and close the cap tightly.

Tube A

Tube B (behind)

REPLACING THE BACTERIOLOGICAL AIR FILTER

Replace the bacteriological filter every 500 cycles or if dirty.

To change the filter:

1. Power the unit OFF, disconnect tube A and remove the filter from the bracket.
2. Disconnect tube B from the back of the filter.

3. Before installing the replacement filter (01-102119S) check that the arrow mark on the filter matches the direction of the arrow on the bracket. (Arrow should be pointing to the right).

4. Insert filter into bracket and connect tube B. Re-connect tube A.

5. After replacing the filter, go to the "Reset LCS Counter" screen in the Setup menu and reset the counter to 0.

LCS system Messages	What it Means	What to Do – User
Replace Water Filter	Water filter in the LCS adaptor is dirty or clogged.	<ul style="list-style-type: none"> • Follow instructions above: "Cleaning the water filter."
Replace Air Filter	The bacteriological air filter in the kickplate is dirty.	<ul style="list-style-type: none"> • Follow instructions above: "Replacing the air filter." • Check LCS adaptor is correctly positioned. • You selected LCS at program start but LCS is not installed.
CF 12 Water Filter Clogged	Problem with Lumens Cleaning System.	<ul style="list-style-type: none"> • Replace Water Filter, see "Cleaning the water filter." • If cleaning doesn't resolve the problem, replace the water filter (01-113742S) • If message persists, call for service.
CF 18 Water Filter Pressure Sensor Stuck	Problem with Lumens Cleaning System.	<ul style="list-style-type: none"> • Call for service.
CF 26 Air Valve error	Problem with air supply pressure in air line. Incoming air pressure must be between 1 bar (15 psi) - 10 bar (145 psi).	<ul style="list-style-type: none"> • Call for service.
CF 30 LCS adaptor disconnected	Problem with Lumens Cleaning System.	<ul style="list-style-type: none"> • Check that LCS adaptor is correctly positioned.

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SciCan Ltd.
Dent4You AG
Coltène/Whaledent GmbH+Co. KG
SciCan GmbH
Coltène/Whaledent AG