




BASIC OPERATION

Follow these steps:

- 1 Power unit ON**
- 2 Press to Unlock Door**
- 3 Load loose instruments in top baskets** provided with unit. Use the hinged basket for small, light items
- 4 Load cassettes in bottom rack** provided with unit. Rack holds up to 10 standard cassettes. If cassettes are not being used, consult the operator's manual for other configurations
- 5 Place Wash Test Indicators on the bottom rack (as required)**
- 6 Close Door**
- 7 Select a Cycle**
- 8 Default drying time is based on a maximum load using STAT-DRI rinse aid. Adjust drying time to suit your clinic's needs.**
Press
- 9 Allow cycle to run to completion**

IMPORTANT!
Remove chemical indicators from cassettes before loading

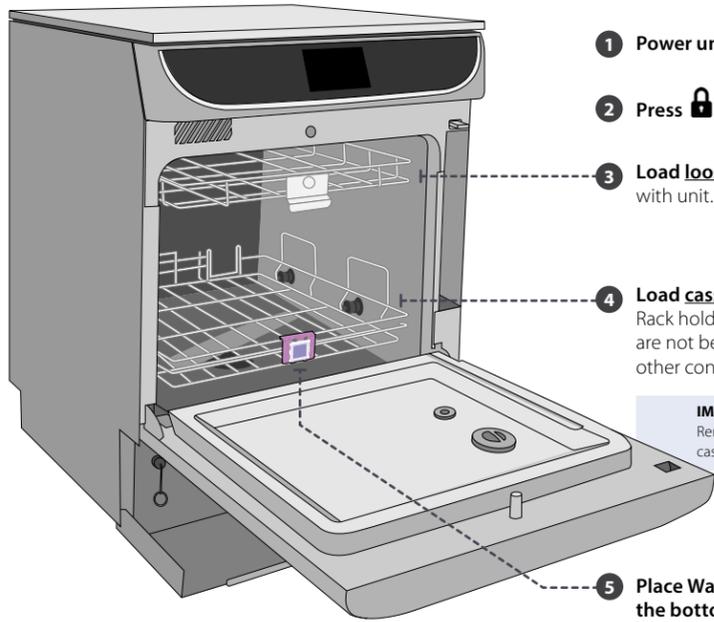

TOUCHSCREEN



Rinse and Hold Cycle

Use to prevent soil from drying on instruments when they will not be washed immediately after loading.



Regular Cycle (Loose Instruments)

Moderately soiled loose instruments.



Heavy Duty Cycle (Cassettes)

Heavily soiled instruments and cassettes.



Door Lock



Cycle Records



Connectivity



Settings



Notifications



ALARM POP-UP GUIDE



Refresh to clear alarm.
If pop-up stays open, more steps are required to fix the issue.



Event Log
for more information.

LED LIGHT GUIDE



WHITE Ready for use



BLUE Cycle is running



GREEN Cycle complete



YELLOW Cycle stopped by user or warning
Flashing condition detected during cycle



RED Cycle fault or alarm condition



ORANGE Software update



PINK Unit being accessed by remote technician




GREY Factory or service mode

OPEN DOOR – NO POWER

1. Open the kickplate.
2. Pull the ring located at the bottom left.

CAUTION! There may be fluid remaining in the unit and the instruments may be hot.

If the cycle was interrupted, instruments should not be used and should be reprocessed.

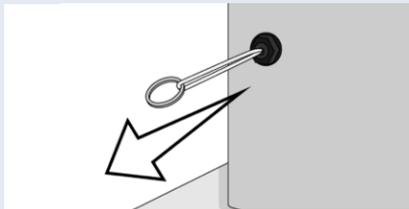

USING WASH TEST INDICATORS

SciCan Wash Test Indicators (WTI) are used for routine monitoring of the cleaning process in instrument washers. They provide a consistent, reproducible method to routinely check the performance of your automated washer process. WTIs are specifically designed for use in HYDRIM units with HIP solution.


Place indicator on the outside edge of the bottom rack (see BASIC OPERATION image above).
Follow local regulatory guidelines for the required frequency of wash test verification.

MAINTENANCE SCHEDULE

DAILY	MONTHLY	EVERY 25 CYCLES	EVERY 750 DRYING CYCLES	EVERY YEAR
Inspect wash arm rotation. Visually check the door seal and trolleys for wear or damage. Check and clean the chamber filters.	Check door lock. Start a cycle and immediately try to open the door. Do not attempt when wash arms are rotating.	Run a cleaning cycle. See CLEANING THE CHAMBER .	Replace the air filter. For video instructions: SETTINGS ⇨ GENERAL ⇨ INSTRUCTIONS ⇨ Replace Air Filter	Annual maintenance by an approved technician.


CLEANING THE CHAMBER FILTER

1. Turn the metal nut at the centre of the filter to release it.
2. Rinse it under a tap and reinstall.

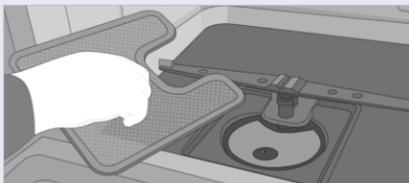

CLEANING THE CHAMBER

1. Pour 750 ml (25 oz) of vinegar or citric acid into the bottom of the empty chamber (with racks) and close the door.
2. Select **SETTINGS** ⇨ **CLEANING** ⇨ **START**

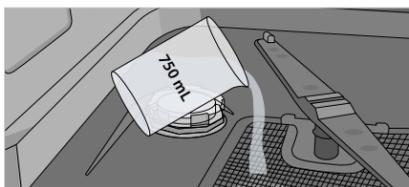

STOPPING A CYCLE

When the STOP button is pressed, the cycle is aborted and cannot be resumed.
Loads interrupted mid-cycle must be fully reprocessed.

1. Press
2. Wait for unit to drain.
3. Press and use gloves to handle instruments.



USING VACATION MODE (for sustained periods of unuse)

If the unit will not be used for more than a week, activate the vacation mode. This will keep any residual detergent in the tubing from drying up and blocking the tubing.



SETTINGS



SYSTEM



VACATION MODE

Follow on-screen instructions.

INSTRUCTIONAL VIDEOS

The HYDRIM has several maintenance and set-up videos to help users learn to care for the unit.



SETTINGS



GENERAL



INSTRUCTIONS

Spare Parts/Consumables

CS-HIPT	HIP Cleaning Solution (4 x 1.75L)
01-112594S	Water Softener Salt
01-108305S	Water Test Kits
WTI-SK	Wash Test Indicator - Starter Kit
WTI-RK	Wash Test Indicator - Refill Kit
8OZPLUS	STAT-DRI PLUS 8oz Bottle with Cap
32OZPLUS	STAT-DRI PLUS 32oz. Refill Bottle
01-113277S	HEPA filter



Terminal sterilization of instruments MUST follow processing in the HYDRIM 112W G4+.



DO NOT put dental burs in the HYDRIM.