


HYDRIM 112W G4+ Quick Reference Guide

Scan for Instructions for Use. PDF reader required.
Full Instructions for Use located at
www.scican.com/products/instrument-washers/HYDRIM-112
For a hard copy, contact SciCan Ltd. at info@scican.com



BASIC OPERATION

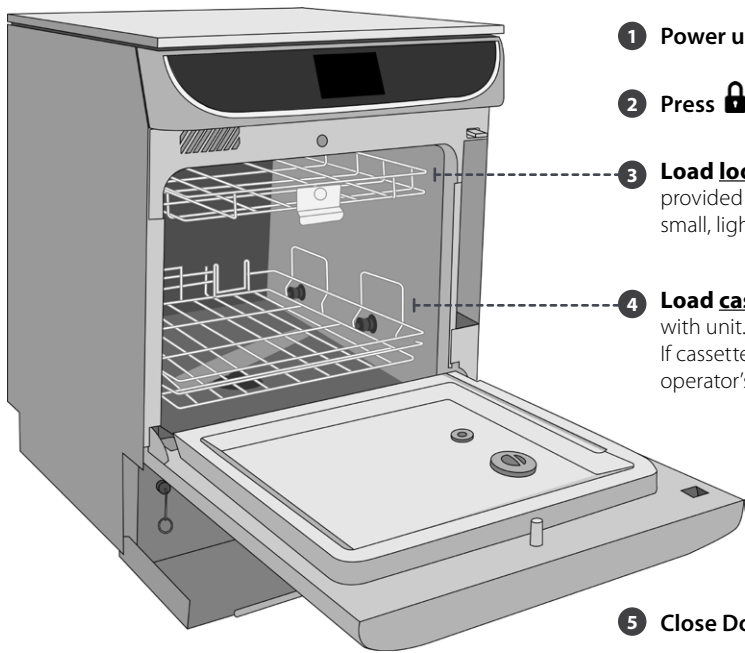

Follow these steps:

- 1 Power unit ON
- 2 Press to Unlock Door
- 3 Load **loose instruments** in top baskets provided with unit. Use the hinged basket for small, light items
- 4 Load **cassettes** in bottom rack provided with unit. Rack holds up to 10 standard cassettes. If cassettes are not being used, consult the operator's manual for other configurations


5 Close Door

6 Select a Cycle

7 Default drying time is based on a maximum load using STAT-DRI rinse aid. Adjust drying time to suit your clinic's needs.

Press

8 Allow cycle to run to completion

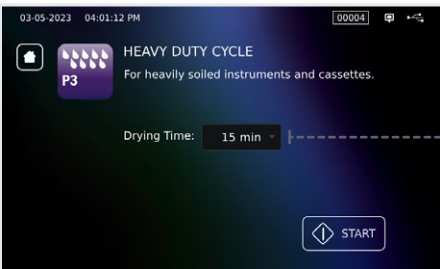

STOPPING A CYCLE

When the STOP button is pressed, the cycle is aborted and cannot be resumed.

Loads interrupted mid-cycle must be fully reprocessed.


1. Press
2. Wait for unit to drain.
3. Press and use gloves to handle instruments.



OPEN DOOR – NO POWER

1. Open the kickplate.
2. Pull the ring located at the bottom left.


CAUTION! There may be fluid remaining in the unit and the instruments may be hot. If the cycle was interrupted, instruments should not be used and should be reprocessed.

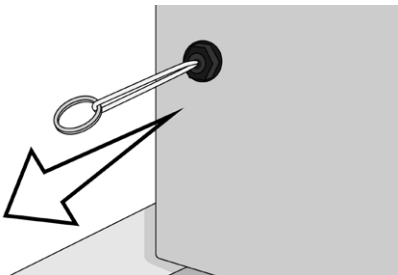

CLEANING THE CHAMBER

The HYDRIM will remind you to run a cleaning cycle every 25 cycles to remove hardwater deposits from the chamber walls and racks. The reminder frequency can be adjusted by a technician.

1. Pour 750 ml (25 oz) of vinegar or citric acid into the bottom of the empty chamber (with racks) and close the door.
2. Select **SETTINGS** **CLEANING** **START**

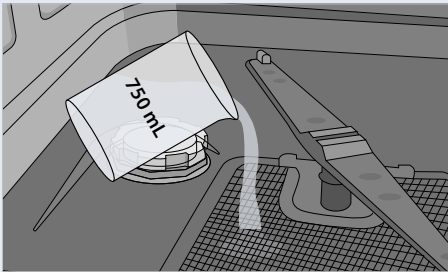


USING WASH TEST INDICATORS

SciCan Wash Test Indicators (WTI) are used for routine monitoring of the cleaning process in instrument washers. They provide a consistent, reproducible method to routinely check the performance of your automated washer process. WTIs are specifically designed for use in HYDRIM units with HIP solution.

NOTE: If using only one indicator, place it in the bottom rack position.

Follow local regulatory guidelines for the required frequency for performing wash test verification.

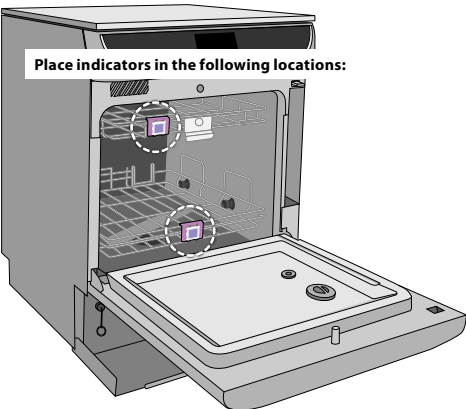

COLTENEGROUP

Manufactured for: **Dent4You AG**
Bahnhofstrasse 2, CH-9435 Heerbrugg
95-117612 CA EN R2. HYDRIM is a registered trademark and HIP is a trademark, of SciCan Ltd., used under license by Dent4You AG.

Distributed by: **SciCan Ltd.**
1440 Don Mills Rd. Toronto, Ontario M3B 3P9 / Canada

www.scican.com

CA 1-800-667-7733
US 1-800-221-3046

TOUCHSCREEN



Rinse and Hold Cycle

Use to prevent soil from drying on instruments when they will not be washed immediately after loading.



Regular Cycle (Loose Instruments)

Moderately soiled loose instruments.



Heavy Duty Cycle (Cassettes)

Heavily soiled instruments and cassettes.



Door Lock



Cycle Records



Connectivity



Settings



Notifications

LED LIGHT GUIDE



WHITE Ready for use



BLUE Cycle is running



GREEN Cycle complete



YELLOW Cycle stopped by user or warning condition detected during cycle



RED Cycle fault or alarm condition



ORANGE Software update



PINK Unit being accessed by remote technician



GREY Factory or service mode

USING VACATION MODE (for sustained periods of unuse)

If the unit will not be used for more than a week, activate the vacation mode. This will keep any residual detergent in the tubing from drying up and blocking the tubing.

START VACATION MODE



SETTINGS



VACATION MODE



Follow on-screen instructions.

STOP VACATION MODE

Press the screen and you will be asked if you are ready to deactivate vacation mode.

YES

Attach detergent cap to detergent bag and press YES.

NO

You can access the LCD while the unit remains in Vacation Mode.

INSTRUCTIONAL VIDEOS

The HYDRIM has several maintenance and set-up videos to help users learn to care for the unit.



SETTINGS



GENERAL



INSTRUCTIONS

Spare Parts/Consumables

CS-HIPT	HIP Cleaning Solution (4 x 1.75L)
01-112594S	Water Softener Salt
01-108305S	Water Test Kits
WTI-SK	Wash Test Indicator - Starter Kit
WTI-RK	Wash Test Indicator - Refill Kit
8OZPLUS	STAT-DRI PLUS 8oz Bottle with Cap
32OZPLUS	STAT-DRI PLUS 32oz. Refill Bottle
01-114390S	HEPA filter



Terminal sterilization of instruments
MUST follow processing in the HYDRIM 112W G4+

SciCan | Your Infection Control Specialist