

DrainDock

Multi-Port Drain Adapter


Save space and simplify drain installations.

DrainDock delivers engineered simplicity through centralized drain management. This compact, multi-port adapter provides a unified drain connection point for sterilizers, automated instrument washers, water filtration systems, and more. It supports up to eight (8) equipment drain lines, consolidating them into one convenient, space-saving adapter.

Easy to install and built to last, this adapter resists corrosion, chlorine, and mineral buildup, ensuring reliable drainage for years of service.



Dimensions

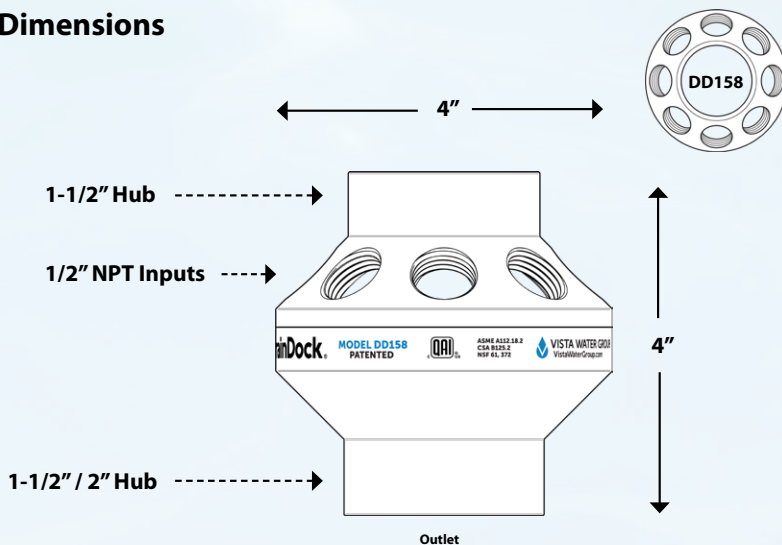

- DD158** DrainDock 8-Port
- DD158FK** DrainDock 8-Port with fittings
- DDFK1** DrainDock Fittings Kit

Required Fittings

Connection	Fitting Part #
STATIM®, STATCLAVE, BRAVO G4, STATCOOL, STATPURE	S4730 (1/4" Male Adapter)
Plug unused ports	S8015 (1/2" Plug)
HYDRIM® (all models) and UC150/300	S8222 (Straight Male Adapter 3/4") & S8322 (Elbow Male Adapter 3/4")
Classic BRAVO	S8214 (Straight Male Adapter 1/4") & S8314 (Elbow Male Adapter 1/4")
STATCLEAN	S8218 (Straight Male Adapter 1/2") & S8318 (Elbow Male Adapter 1/2")


Materials

- › Schedule 40 PVC
- › 1/2" NPT threaded inputs (2, 4 or 8)
- › Built-in output coupling accepts 1-1/2" pipe or a 2" coupling
- › 100% lead-free for potable water environments

Standard Certification & Compliance


- › QAI-Certified Product
- › ASME A112.18.2 / CSA B125.2
- › ASTM D2665, F1498
- › NSF 61, 372

DrainDock Position Under Sinks

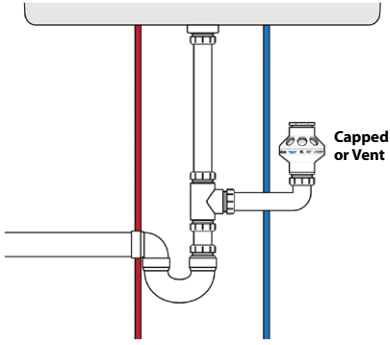


Method A

Installation beside the main drain line can offer maximum flexibility and can be achieved with the use of a simple drain tee adapter fitting.

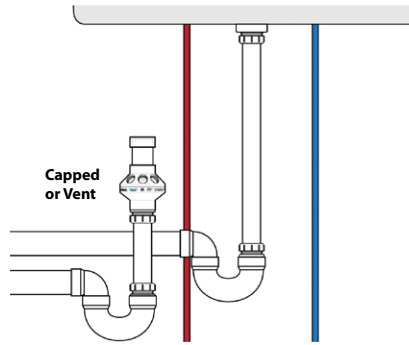

Method B


When possible, as in cases of new construction, it can be beneficial to run parallel drain waste systems to separate waste from the sink from other appliances and devices.

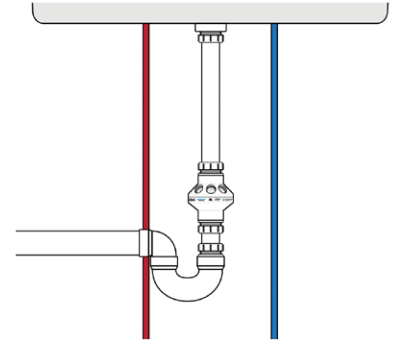


Method C

When necessary, it is possible to install the DrainDock fitting directly in the main drain line under a sink. However, this position can result in more audibly noticeable drainage of waste containing a significant amount of air.


Connection to Waste Lines

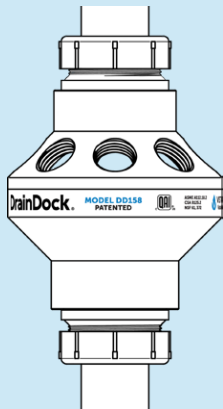

Can Be Connected with Trap Adapters

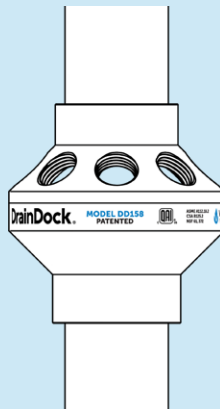


1-1/2" Pipe Fits Inside Hubs

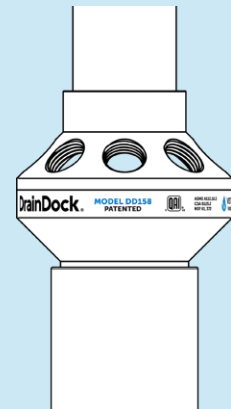

Outlet Hub Fits Into 2" Coupling for 2" Discharge Port

A variety of compatible 1/2" NPT threaded push-to-connect, compression, and hose barb adapter drain fittings are available.

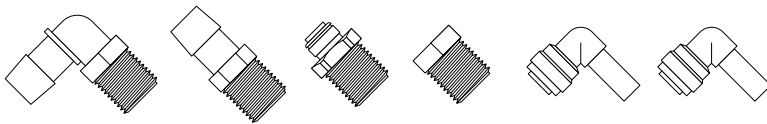

IMPORTANT

Always install a 1/2" NPT threaded plug in any unused port(s) of the DrainDock, and always follow applicable local plumbing codes.

For more information, visit www.scican.com

Manufactured by:
Vista Water Group
1244 CR 1475,
Ashland, OH
44805, USA

Distributed by:
SciCan Ltd.
1440 Don Mills Rd
Toronto, ON
M3B 3P9, Canada

Distributed by:
Coltene/Whaledent Inc.
235 Ascot Pkwy.
Cuyahoga Falls, OH
44223, USA

DrainDock is a registered trademark of Vista Water Group.
SciCan is a Coltene Group company. STATIM and HYDRIM are registered trademarks; STATCLAVE, BRAVO and STATCLEAN are trademarks of SciCan Ltd.
STATPURE and STATCOOL are trademarks of Coltene/Whaledent AG.

SciCan | Your Infection Control Specialist